



Geberit AquaClean Mera CARE

Shower Chair / Equipment Compatibility Report

5th edition

Author – Simon Thomas

Updated – July 2024

Index

Segments

Product overview

Shower chairs

Tilt in Space shower chairs

Paediatric equipment

Bariatric equipment

Toilet lifters

Miscellaneous Equipment

Product details & Manufacturer contact details

How to access shower chair mode



Rimless ceramic – TurboFlush

App control

Wireless Opto remote

User remote control

Self servicing function

5 Year FREE Warranty

YouTube video support



Constant warm water

Industry leading **1 minute** Wash cycle

Whirlspray Shower Technology

Oscillating Wash

Adjustable wash arm positioning

5 programable user settings

Bluetooth Technology



Compatibility

With an estimated 50% of wash dry toilets provided under a DFG grant being used with a type of shower chair or assistive equipment, compatibility is essential.

This report looks at the wide range of equipment available on the healthcare market that works satisfactorily with the **Geberit AquaClean Mera Care** wash dry toilet.



Shower Chairs

Attendant or self propel, there is a wide range of upright shower chairs available from many manufacturers.

The Geberit AquaClean Mera CARE does not require an engineer to adjust, adapt or bypass functions to enable compatibility post installation

Watch the following video for more information relating to compatibility and necessary considerations when using a shower chair with a shower toilet.

Link – [Shower Chair Compatibility Video](#)



Osprey Healthcare

Attendant Sani Commode

Key Features

- Stainless Steel (Rust Resistant)
- Adjustable Seat Height 19"-22"
- Flexi Seat, Horseshoe Seat,
- Toilet Seat, Various apertures
- Detachable Foldaway Footrest
- Foldaway Padded Arms
- Compatible with Washer/Dryer toilets
- 30 Stone SWL



Chiltern Invadex

Aqua Master A06G Attendant

Key Features

- Safe working load 200kg (31st) (440lb)
- The chair features a stainless steel frame with corrosion resistant wheels and castors.
- Several footrest options are available, each is height adjustable to ensure that the user is in a comfortable position.
- Seats are available as flat padded, full ring aperture or horseshoe with a range of aperture sizes to suit the user.
- 18", 19" and 21" chairs are Geberit compatible.
- Attendant propelled models are provided with four braked castors, directional locking castor available on request.



Care & Independence

Options 7710C Shower Chair

Key Features

- Adjustable height – from wheel to seat
- Different options of footrests – detachable wheelchair style footrest or a one-piece footrest
- Four armrest options
- Can be used over a toilet or with a commode pan
- Safe Working Load of up to 200kg
- Different wheel options available – transit, attended and self-propelled
- 15-year warranty on frame




Chiltern Invadex

TransAqua Attendant

Key Features

- Safe working load 200kg (31st) (440lb)
- The chair features a stainless steel frame with corrosion resistant wheels and castors.
- Armrests and footrests swing away to aid transfer.
- Several footrest options are available, each is height adjustable to ensure that the user is in a comfortable position.
- Seats are available as flat padded, full ring aperture or horseshoe with a range of aperture sizes to suit the user.
- Self-propelled models can be provided with 20", 22" or 24" wheels.
- Attendant propelled models are provided with four braked castors, directional locking castor available on request.

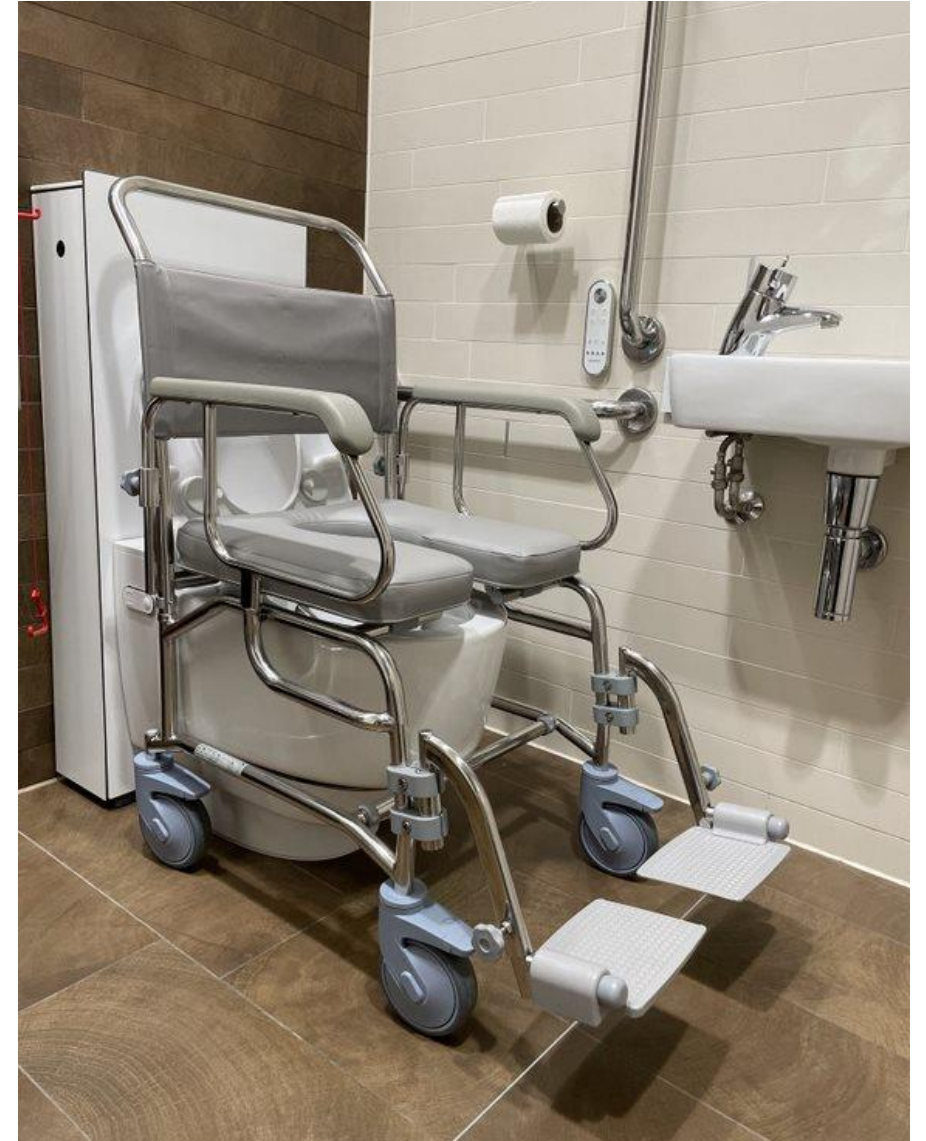

Freeway (Prism Medical)

Freeway T40 Auto

Key Features

- Available in three standard sizes, based on seat width:
- 'A' - 540mm or 21"
- 'B' - 490mm or 19"
- 'C' - 440mm or 17"




Wealden Rehab

The Raz AP GEB Shower Chair

Key features

- Attendant Propelled
- Stainless steel height adjustable frame
- 25 stone weight limit
- Seat depth – 400 or 480mm
- Overall length with footplate – 860mm
- 5 Year warranty
- Tension adjustable back rest

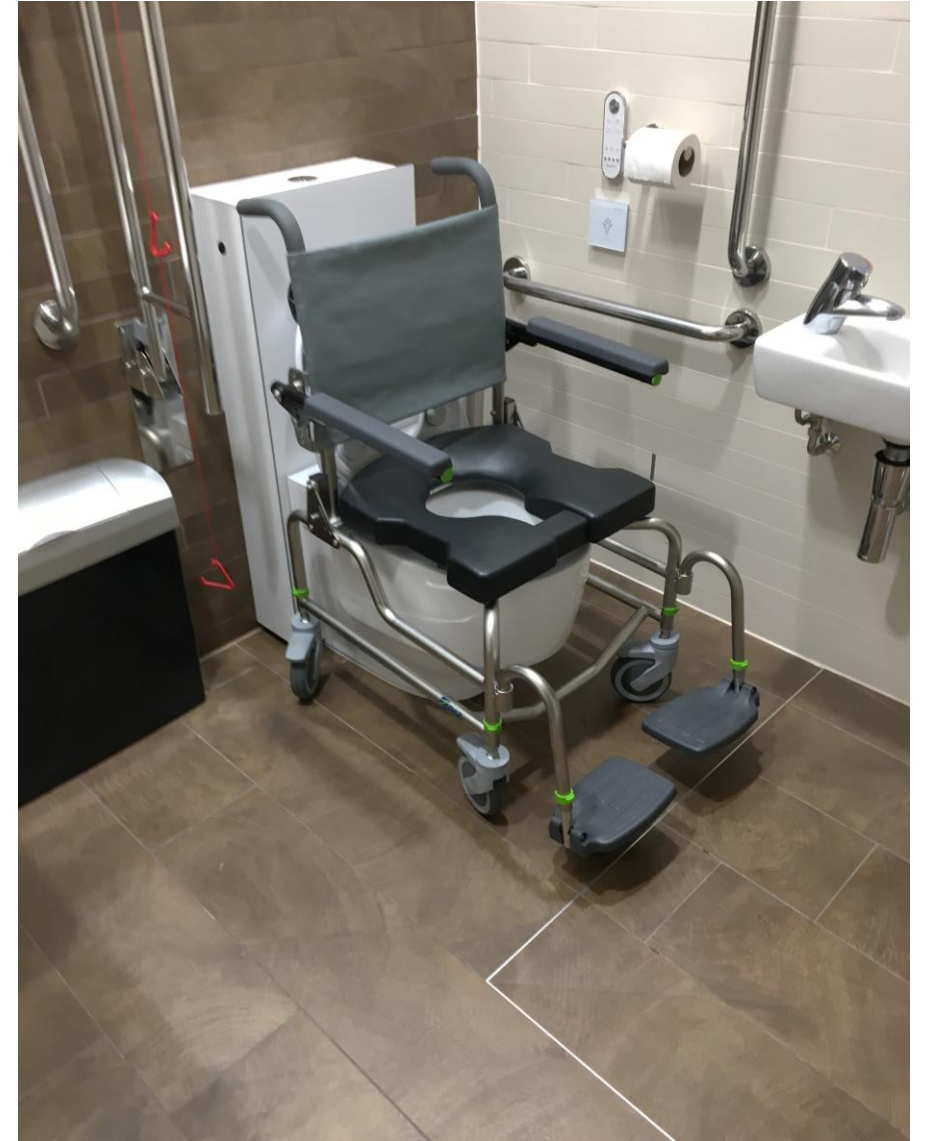

Invacare

Aquatec Ocean shower chair

Key Features

- Stainless steel frame
- Machine washable back rest
- Adjustable height seat settings
- Horseshoe Seat
- Fold away footplates
- Maximum user weight 180kgs



ETAC

Etac Swift Mobil - 2

Key Features

- Attendant wheels
- Adjustable Seat Height (45 – 60cm)
- Adjustable Back Rest
- Ergonomic Footrest
- Etac Swift Mobil-2 XL for larger users
- 160kg User Weight Limit



ETAC

Etac Clean Shower Commode Chair

Key Features

- Unique foot support
- Swing away arm rests
- 130kg Safe working load
- Range of accessories



Osprey Healthcare

Bath Mobile Shower Toilet Chair

Key Features

- Easily dismantled
- Rust proof
- Flip away arms
- 130kg Safe working load
- Range of accessories



Drive Devilbiss

Aston Heavy Duty Shower Commode

Key features

- Seat width 56cm (22")
- Seat depth 43cm (17")
- Tool-free height and angle adjustable footplates
- Simple e-clip height adjustment to suit different toilet heights
- Armrests flip up for side transfers
- Rust proof aluminium frame
- **MAXIMUM WEIGHT CAPACITY 260kg (40st)**

*NB. Geberit recommend use only on a short term basis / emergency use.



Smit Care

SmitCare Apex Mobile Commode (SAC360)

Key features

- Four braked castors
- Removable arms and swing-away, lift off footrests
- Seat height 500mm (19.5")
- Seat size 435 x 460mm (17 x 18")
- Castors 130mm (5") diameter
- 30 stone (190kg) max user weight limit

*NB. This product may not be stainless steel and may rust. Geberit recommend use only on a short term basis / emergency use.



Osprey Healthcare

Self Propel Sani Commode

Key Features

- Stainless Steel (Rust Resistant)
- Adjustable Seat Height 19"-22"
- Flexi Seat, Horseshoe Seat,
- Toilet Seat, Various apertures
- Detachable Foldaway Footrest
- Foldaway Padded Arms
- Compatible with Washer/Dryer toilets
- 30 Stone SWL




Wealden Rehab

The Raz SP GEB Shower Chair

Key features

- Attendant Propelled
- Stainless steel height adjustable frame
- 25 stone weight limit
- Seat depth – 400 or 480mm
- Overall length with footplate – 860mm
- 5 Year warranty
- Tension adjustable back rest

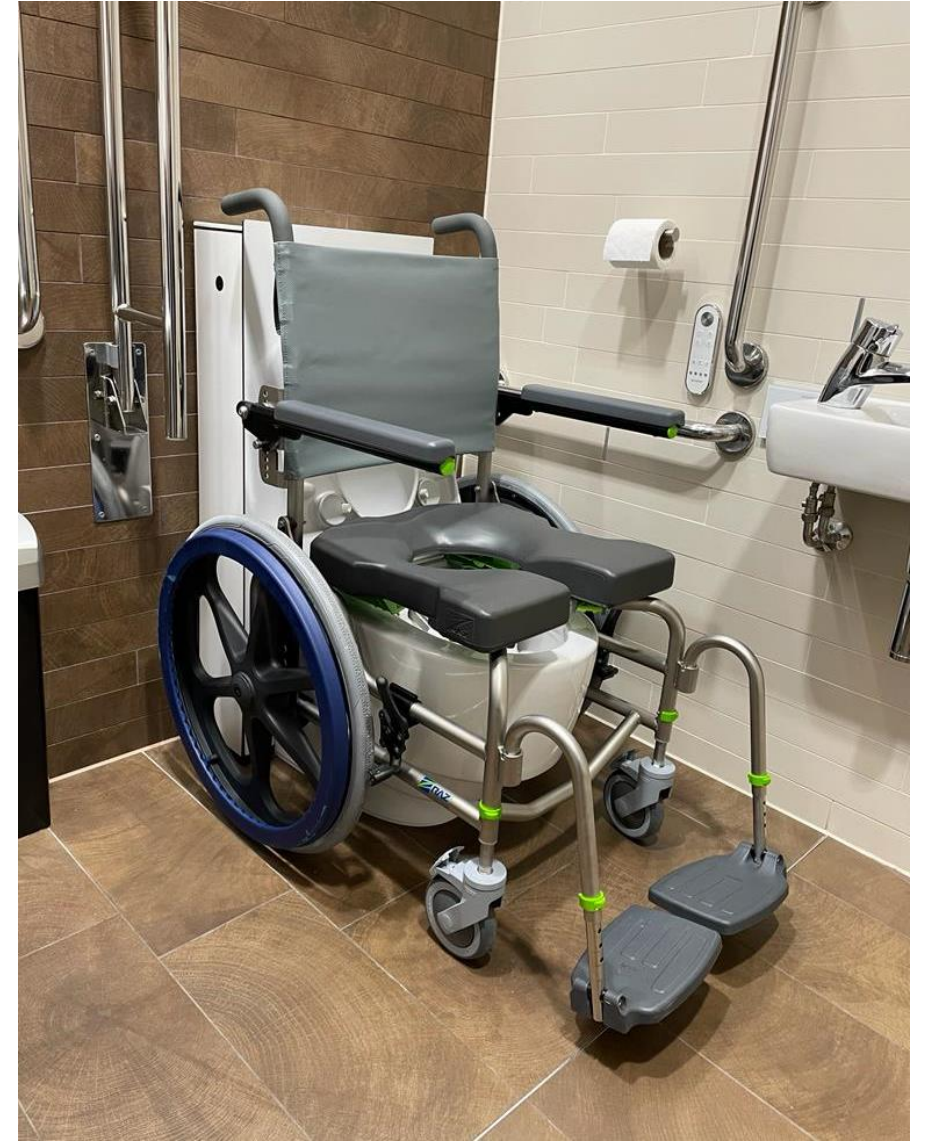

Care & Independence

Options 7710C Shower Chair Self Propel

Key Features

- Adjustable height – from wheel to seat
- Different options of footrests – detachable wheelchair style footrest or a one piece footrest
- Four armrest options
- Safe Working Load of up to 200kg
- Different wheel options available – transit, attended and self propelled
- 15 year warranty on frame
- Padded seat options with skirts available in horse shoe or full aperture



Chiltern Invadex

Aqua Master A11G Self Propelled

Key Features

- Safe working load 200kg (31st) (440lb)
- 17, 19 and 21” seat width options
- Rigid one piece stainless steel painted base frame.
- Removeable armrests and footrests swing away to aid transfer.
- Several footrest options available.
- Seats are available as flat padded, full ring aperture, horseshoe or side access with a range of aperture sizes to suit the user.
- Options include extended seats and pressure relief foam.
- Self-propelled models can be provided with 20”, 22” or 24” wheels.



Chiltern Invadex

TransAqua Self Propelled

Key Features

- Safe working load 200kg (31st) (440lb)
- The chair features a stainless-steel frame with corrosion resistant wheels and castors.
- Armrests and footrests swing away to aid transfer.
- Several footrest options are available, each is height adjustable to ensure that the user is in a comfortable position.
- Seats are available as flat padded, full ring aperture or horseshoe with a range of aperture sizes to suit the user.
- 19" and 21" chairs are Geberit compatible. **(Not 18")**
- Self-propelled models can be provided with 20", 22" or 24" wheels.



ETAC

Etac Swift Mobil 24"-2

Key Features

- 24" Self-Propelled Wheels
- Adjustable Seat Height (45 – 60cm)
- Adjustable Back Rest
- Ergonomic Footrest
- 160kg User Weight Limit



Freeway (Prism Medical)

Freeway T70 Auto

Key Features

- The Freeway T40 / T70 AUTO is for use with automatic toilets. Features pram handle backrest and seat with skirt as standard.
- The Freeway T40 Shower Chair range is available in three standard sizes, based on seat width:
 - 'A' - 540mm or 21"
 - 'B' - 490mm or 19"
 - 'C' - 440mm or 17"



ETAC

- **Etac Clean 24" self propelled**

Key Features

- Unique foot support
- Swing away arm rests
- 130kg Safe working load
- Range of accessories



Freeway (Prism Medical)

Freeway T60A / T60B

Key Features

- Safe Working Load (SWL): 200kg (30 stone)
- Available as a shower, toilet or commode chair
- Perfect for use in tight spaces
- Anti-tip bars fitted to the rear as standard
- Pliable backrest moulds to the user while providing excellent support
- FREECOTE®, antibacterial and antimicrobial coating.
- **19” or 21” seat widths only as standard**



Osprey Healthcare

Flyta Active shower chair

Key features

- Stainless steel frame
- 17" or 19" Seat widths
- 15 Year warranty
- Toilet, Horseshoe, or full aperture Seat options available
- 24" Self propel wheels
- Webbed calf rest
- 18 stone user weight limit



Tips

Always specify that the shower chair is for use with the Mera CARE shower toilet at time of ordering

AquaMaster Range
install toilet at 410mm seat height

Care & Independence
self propel – install at 435mm CERAMIC height and shower chair on its 2nd hole down to achieve the lowest shower chair seat height possible

All standard Freeway “Auto” Chairs
- install toilet @465mm seat height

Etac Range of chairs
Geberit recommend using the Soft Comfort Seat height 2cm Cushion

Tilt In Space Shower Chairs

Tilt in Space shower chairs have gained in popularity. Offering safe & secure support for both toileting and showering.

Osprey Healthcare, Freeway (Prism Medical), Care and Independence, Chiltern Invadex all have a range of upright shower chairs that are compatible with the Geberit AquaClean Mera CARE shower toilet.



Care & Independence

Adult Tilt in Space Chair

Key Features

- Adjust anywhere between upright and 45° to best suit your requirements
- Accessories include harness, thoracic support & hip straps
- Frame is made from stainless steel which helps with recycling
- Thoracic lateral support and lateral leg support available
- Safe Working Load of 150kg
- 15 year warranty on frame
- Padded seat options with skirts available in horse shoe or full aperture



Osprey Healthcare

981 Tilt in Space Chair

Key Features

- Easy to Tilt
- Adjustable Headrest
- Adjustable Footrest
- Supportive Mesh
- Optional Mesh Aperture
- Stainless Steel (Rust resistant)
- Adjustable Seat Height



Wealden Rehab

The Raz ART Shower Chair

Key features

- IPAS Seat System - adjust the seat depth to achieve optimum aperture positioning
- Rotational Tilt frame design
- Stainless steel chassis for durability
- Height adjustable
- Tension adjustable backrest
- Tilt-in-space for excellent pressure management and comfort
- Flip-up armrests for access



Wealden Rehab

The Raz AT Shower Chair

Key features

- Stainless steel chassis for durability
- Height adjustable seat
- Tension adjustable backrest - supports posture and provides lateral support
- Tilt-in-Space to 40° - excellent for pressure management and comfort
- Flip-up armrests for excellent access
- IPAS Seat System - adjust the seat depth to achieve optimum aperture positioning



Freeway (Prism Medical)

T80 Geberit Mera CARE Auto

Key Features

- Easy to Tilt
- Removable Headrest
- Adjustable Footrest
- Freecote protective frame covering
- Wide range of positioning accessories



Chiltern Invadex

Aqua Master Powered Tilt in Space Shower Commode Chair

A16G – 18” seat width

A17G – 21” seat width

Key Features

- Full range of aperture, horseshoe and flat padded seats available
- Option for bespoke seating, including pressure relief foam
- Armrest, footrest and backrest options
- Extensive range of accessories available
- Corrosion resistant stainless steel and nylon braked castors
Directional locking option available
- Stainless steel painted frame
- Electric Version available
- Modified frame available for side exit WC waste pipe



Invacare

Ocean VIP Ergo

Key Features

- Stainless steel frame
- -5° to 40° seat tilt angle
- 480mm Seat width
- Modular Design
- Tilt in space
- Height Adjustable - 510mm – 610mm



Etac

The Swift mobil Tilt - 2

Key Features

- Unique tilt action
- Adjustable backrest
- Height adjustable
- Head support
- Head Support
- Tool free adjustments



R82

The Heron manual TILT In Space

Key Features

- Unique tilt action
- Adjustable backrest
- Height adjustable
- Head support
- Head Support
- Tool free adjustments



Tips

Freeway T80 Auto
- install toilet at
465mm seat
height.

Always specify with
Freeway to be used with
Mera CARE shower toilet

Install toilet at
410mm ceramic
height for all
standard height
Chiltern Invadex
Chairs

Osprey
Healthcare
TIS is height
adjustable to
match Geberit
seat heights

The RAZ ART, the
Invacare ERGO VIP
can work with side
exit soil pipe

Care &
Independence
Adult Tilt In
Space is height
adjustable to
suit any Geberit
seat height

Always specify
Tilt in space
shower chair is
for use with a
shower toilet
when ordering

Paediatrics

Paediatric equipment has rapidly improved in recent years, offering greater support and helping to promote independence in a safe environment

Osprey Healthcare, Freeway (Prism Medical), Rifton and R82 are some of the market leaders in Paediatric shower chairs.

Other more basic equipment is also available via some high street stores and internet sellers.



Freeway (Prism Medical)

T90 Paediatric Self Propel Shower Chair

Key Features

- 365MM seat width
- Available as assistant or self-propelled models
- Integral horizontal push handled backrest
- Pan sliders attach to the frame and are easily removable without tools
- Supplied as standard with straight paediatric armrests and small horseshoe seat
- Different height armrest sockets
- Different positions for armrest sockets
- Clearance on this frame is now 475mm – Install Geberit @ 465mm



Freeway (Prism Medical)

T80 Paediatric Reclining Shower Chair

Key Features

- Easy to Tilt
- Removable Headrest
- Adjustable Footrest
- Freecote protective frame covering
- Wide range of positioning accessories



Jiraffe

HTS (Hygiene Toileting System) by Rifton

Key Features

- Three seat sizes
- Tilt in Space
- Unique seat pad design
- Load bearing footplate
- Thoracic Supports
- Height Adjustable
- Seat depth Adjustable



R82

Flamingo

Key Features

- Four different sizes
- Back angle adjustment
- Height adjustable
- Tilt in Space
- Adjustable footplates
- Side Supports
- Height Adjustable
- Head Support



Moorings Mediquip

Hoggi Sharky

Key Features

- 45° tilt for high safety
- Simple and practical use
- Very compact folding size (even with attached seat)
- Extremely slim, removable seat offers water-saving use in the bathtub
- Easy and comfortable height adjustment
- Various seat sizes



Argos / Amazon / Online

Children's toilet seat with step

Key Features

- Quick and affordable
- Overnight delivery
- Ideal for young children and individuals with Dwarfism
- Available in a wide range of colours



Tips

Rifton HTS & R82 Flamingo work best with seat up – contact Geberit area manager at time of install for guidance

Children's toilet seat with step – Install Geberit at 430mm

Install Geberit at 465mm for FreewayT80 Paediatric

All sizes of the Rifton HTS work over a Mera CARE

Do not use the Flamingo Hi – Low base with Mera CARE. Small seat base does not fit

Bariatrics

With an increase in awareness of the needs of larger people and the adaptations needed for existing equipment to support a much larger body weight there has never been such a greater focus. Equipment is now available to suit all Plus size, Mediatic and Bariatric users.

Equipment is now on the market capable of supporting users up to 85 stone



Osprey Healthcare

The Java Bench

Key Features

- Choice of Toilet or Horseshoe seat
- Skirt Included for Geberit compatibility
- Safe working load of 85 stone
- Slight incline on the seat to aid with standing & comfort
- Also compatible with other wash/ dry and standard toilets
- 18, 20 or 22” seat height
- Choice of 2 leg styles, bolt down or rubber ferrules.
- 15 Year frame warranty
- Click here for more information - [Shower toileting for Bariatric Users - YouTube](#)



Freeway (Prism Medical)

Bariatric AUTO Static Bench

Key Features

- 55 Stone safe working load
- 19", 21", 23", 25", 27", 29" 31", 33" Seat Widths
- bespoke wider sizes if required.
- Seat types: Horseshoe or Aperture
- Removable Arms



Langhams

Gordon Ellis Seat & Frame Horse Shoe

Key Features

- Comfortable moulded plastic toilet seat
- Seat easily clips on and off the frame for cleaning
- 30 Stone weight limit



Langhams

Gordon Ellis Seat & Frame Full Aperture

Key Features

- Comfortable moulded plastic toilet seat
- Seat easily clips on and off the frame for cleaning
- 30 Stone weight limit



Aidapt Sussex Bariatric Commode

Key Features

- Vinyl padded over seat supplied for discreetness when not in use
- 7.5 litre commode bucket and lid provided as standard
- The Sussex commode ideal for users with limited mobility
- Can be used as a standalone commode or can be positioned over most WC suites
- Height adjustable legs and support arms as standard
- Maximum user weight limit 254kg (40 stones)



Tips

Removal of Seat gives better clearance for the “Shelf” and allows for full functional use of the wash and dry functions

Osprey seat height of 19” requires a Geberit seat height of 445mm or a ceramic height of 430mm

Langham Multi frame also available with a big john seat attachment

Removal of toilet seat requires a Free of charge visit from a Geberit area manager

Toilet Seat Raisers

Toilet seat raisers have been in circulation for many years and have become more popular in recent times due to an increase in manufacturers reducing cost.

Based on the principle actions of a riser recliner chair a seat raiser can help the user to go from sit to stand and vice versa with ease. For those with more complex needs a verticle lift option is also available.



Osprey Healthcare

The Osprey toilet Raiser

Key Features

- Installs Anywhere
- Free-standing unit fits over any toilet
- No assembly/tools required – simply unpack
- Comes with AC plug and rechargeable battery pack
- Easy to Use
- Height Adjustable
- Stainless Steel frame for stable support
- Back Rest – optional extra
- Raises and lowers slowly and smoothly
- Click here for more information - [The Osprey Healthcare Toilet Lifter - YouTube](#)



Ropox

ROPOX Toilet Lifter

Key Features

- Slim, modern profile
- 524mm wide x 205mm deep x 1154mm height (in lowered position)
- Utilised Quality Geberit components
- Height adjustable
 - Electric – 25cm height adjustment (40 to 65cm seat height)
 - Manual – 22cm height adjustment (40 to 62cm seat height)
- Range of optional extras including support arms.



Pressalit

Select TL2 Toilet Lifter with Side Profiles

Key Features

- Electrically height adjustable 400 mm (425-825 mm) with wired remote control.
- Different heights to suit different users/transfer techniques
- Manual or powered adjustment to suit different requirements - frequent/periodic adjustment
- Allows a range of support arms, back rests and WC's to be fitted for a bespoke solution



Aerolet

Aerolet sit to stand

Key Features

- Max user weight 180Kg
- Simple to use controls
- Incorporates Geberit own toilet seat for zero change to function
- Seat seal under frame ensure optimum drying



Aerolet

Aerolet Vertical lifter

Key Features

- Max user weight 180Kg
- Simple to use controls
- Incorporates Geberit own toilet seat for zero change to function
- Seat seal under frame ensure optimum drying



Tips

The Osprey Healthcare Toilet Lifter works with the toilet seat down or taken off. The skirt on the Lifter skims the ceramic when set up correctly

The Ropox toilet lifter with support arms can come with a Ropox made bracket to hold the Geberit remote control.

Other equipment

There are many basic products that fit around the toilet offering various levels of support or assistance. Often non specialist equipment can be as suitable

Drop down, wall mounted arms, toilet frame surrounds, MD frames are some of the most common accessories.



Toilet surround frames

Padded Back support

Key Features

- Quick and easy to install
- Can be set at height user requires
- DDA Compliant
- High quality stainless steel tube
- Concealed Fixings
- Easy clean white padded back / head rest
- Max projection: 310 mm. Max width: 542 mm
- <https://www.prestigewashrooms.co.uk/prestige-stainless-steel-padded-backrest-542mm.html>



Squatty Potty

Squatty Potty

Key Features

- mimics natural squat posture
- slightly raised heel for comfort
- wide platform for natural posture & stability
- opens the colon for fast, easy elimination
- Stores perfectly on Mera CARE seat heights 46cm or above
- Works well with all seat heights.



Gel Ovations

The Gel Seat

Key Features

- Offers Pressure relief whilst toileting.
- Gel filled centre
- Quick and easy to remove
- Covered in waterproof Dartex material
- Non slip underside for safe transferring



Tobii Dynavox

TD Pilot – Eye Gaze Technology

Key Features

- Multiple Access Methods
- Partner Window
- Outdoor Eye Tracking
- Powerful Speakers
- Meets Apple Performance Standards
- Fully Compatible with Geberit Home APP



Eye tracking for iPad

TD Pilot is an eye-controlled communication device for iPad. Designed to empower people with conditions such as ALS/MND, spinal cord injury or cerebral palsy to communicate and use their favourite apps, this iPadOS-based speech generating device features the world's leading eye tracker.



Toilet surround frames

Frames

Key Features

- Quick and easy to place
- Various manufacturers
- Stock Loan store item
- Height adjustable




Wall mounted drop down rails

Frames

Key Features

- Quick and easy to place
- Various manufacturers
- Available with in built toilet roll holder
- Various style and coatings

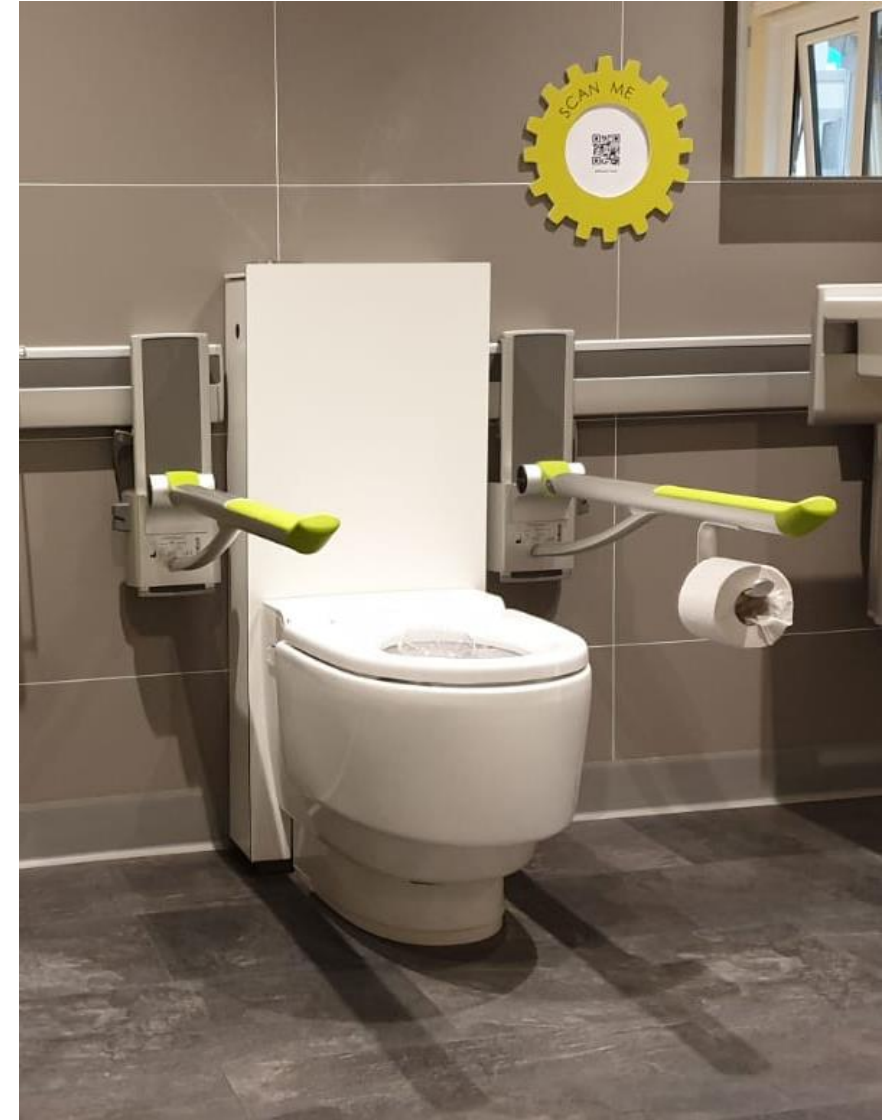

DoAbility

Easyfold Mobile Shower Commode

Key Features

- Aluminium Frame
- Blue PVC & Polyurethane Upholstery
- Removable Seat & Cover for Easy Cleaning
- Soft PU Non-Slip Armrest
- 4 Locking Castors
- Removable Bucket (front removal)
- Retractable Footrest
- Shower cover for seat
- Foldable for Easy Storage and transport



Daily Care

The MD Frame

Key Features

- Installs Anywhere
- Free-standing unit fits over any toilet
- Screws to floor for stability
- Arm rest options
- Easy to Use
- Adjustable height arms and footplate



Standard Specification

- ✓ **5 year FREE Warranty**
- ✓ 5 Profile settings
- ✓ Oscillating Wash
- ✓ Adjustable spray position
- ✓ Constant warm water
- ✓ Opto remote
- ✓ User remote control
- ✓ App enabled
- ✓ Lady wash
- ✓ Height adjustable
- ✓ Self servicing function

Optional Extras

- Toilet seat Lid (Product codes 243.211.11.1 & 243.404.00.1)

NEW

GEBERIT

GEBERIT CARES

GEBERIT AQUACLEAN CARE.
INNOVATIVE THINKING
FOR INDEPENDENT LIVING

With over 40 years experience in developing shower toilet technology, Geberit has invested all their wealth and expertise to offer something new and innovative. The new AquaClean Care shower toilet has been specifically designed to improve quality of life for those elderly and disabled people who have difficulty using a conventional toilet, to create the perfect combination of aesthetics and functionality.

5 YEAR WARRANTY
5 years parts and labour when you register your product

30 Stone User Weight Limit



Manufacturer links

www.theospreygroup.co.uk
www.prismmedical.co.uk
www.chilterninvadex.co.uk
www.jiraffe.org.uk
www.r82.com
www.careandindependence.com


www.dailycare.co.uk
www.langham-ge.co.uk
www.invacare.co.uk
www.gelovationseurope.com
www.tobiidynavox.com



HOW TO SELECT ACCESSIBILITY (SHOWER CHAIR) MODE

Rear of remote control-LCD Panel and Cursors


1. Battery Compartment

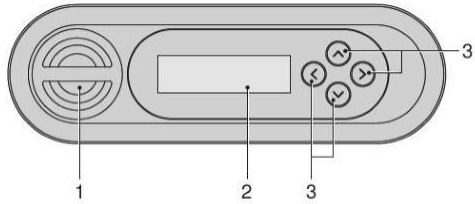

2. LCD screen

3. Navigation / Selection Cursors


Step 1

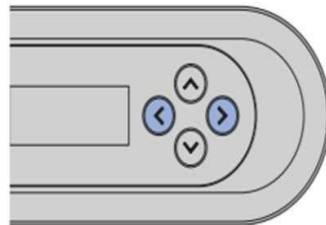

Press Left/right cursor to launch screen, then scroll left/right until you see :

Care and Maintenance


Step 2

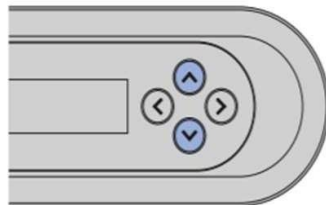

Scroll down until you see:

Accessibility Mode
< off on >


Step 3

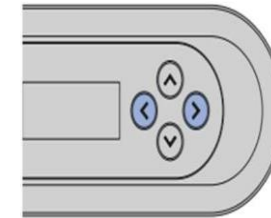

Scroll Left/Right until **on** is underlined:


Accessibility Mode
< off on >

Scroll down until you see:

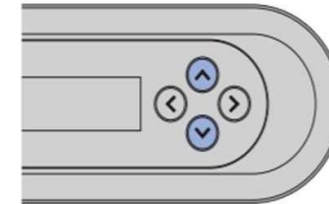

Activate Mode?
< off on >

Step 4

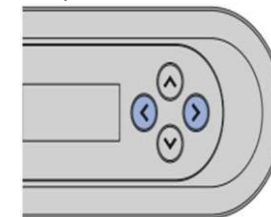

Scroll Left/Right until **on** is underlined:

Activate Mode?
< off on >

Scroll down to
save